
NSK

Tip for periodontal membrane removal

G95

OPERATION
MANUAL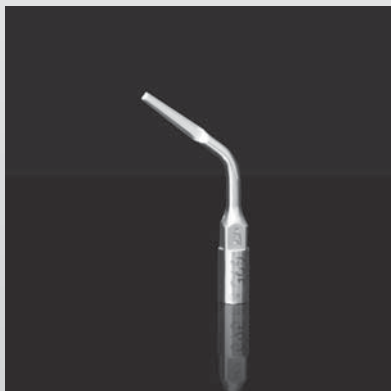

CE 0197 MADE IN JAPAN


OM-Z0234E

Thank you for purchasing this product. This tip is for incision of the parodontium or removal of the periodontal membrane. Before use, make sure to read this instruction manual carefully and keep it in a place where it can be referred to at any time.

**CAUTION**

- This tip is specifically for the NSK "Multi Function Ultrasonic Scaler Varios Series." Do not use it with other systems.
- This tip should be used by qualified personnel for the purpose described in the "Application" section. Do not use it for other purposes.
- Make sure to set the output in accordance with "3-2 Output setting" in the section "3 How to use."
- Make sure to use under sufficient water injection. Insufficient water injection may cause burning of the affected area or overheating of the equipment.
- Before use, make sure to check that the output setting is correct.
- Before use, make sure to check that the tip is securely attached to the handpiece.
- The tip is a consumable. Replace the original tip with a new tip promptly in accordance with "Time for tip replacement" in the "Maintenance" section. Also, replace the tip with a new tip on a regular basis because, in time it may fracture due to metal fatigue.
- Should the tip break during use, immediately stop operation and replace the tip with a new tip. Parodontium, etc., may be damaged by the broken part.
- Do not grind or bend the tip. The tip may fracture or fail during vibration.
- Do not use the tip with a flaw, bend, corrosion, etc. The tip may fracture or fail during vibration.
- Do not use on patients who use a cardiac pacemaker.
- This product is non-sterile. The tip must be sterilized before use.

1 Product name and product number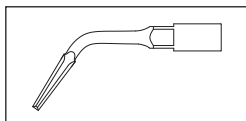

Product name: G95

Product number: Z217705

2 Features

- Access to a surgical site is improved by thinning the tip apex shape and sharpening both edge flanks.
- The tip is TiN (titanium) coated to improve surface glide.
- Adhesion areas of periodontal membrane can be removed smoothly.
- Tooth removal can be performed smoothly so that patient discomfort can be reduced.
- Approaches from angles hard to reach with hand instruments are also possible.
- Tooth removal can be performed while suppressing damage to the cavity wall, unlike an elevator.
- Excessive ablation of bone is reduced, and conservative ablation of alveolar bone is possible.

3 How to use

3-1 Installation

- 1) First securely screw the tip by hand into the handpiece until it is tight (Fig. 1).
- 2) Locate the tip wrench aligning with the square portion of the tip. Next, tighten the tip wrench (anticlockwise) until it turns freely and clicks (Fig. 2). At that time, hold the handpiece so as not to kink the handpiece cord.

To remove the tip, use the tip wrench and turn (anticlockwise) in the "Loosen" direction in Fig. 2.

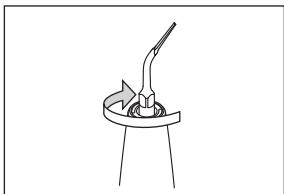
Fig. 1

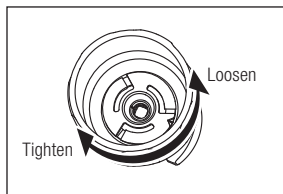
Fig. 2

**CAUTION**

- If the tip is installed with any contamination adhering to its threaded portion, vibration may weaken or fail. Clean the threaded portion if contamination adheres to it.
- If you feel no vibration with the tip during use, stop operation and remove handpiece from the patient's oral cavity. Then reactivate the handpiece again. If the situation does not resolve, the tip may have become loose during use. Recheck the installation of the tip.
- When installing the tip onto the handpiece wear sterile gloves, and use an individually sterilized tip, handpiece, and tip replacement wrench.
- The tip replacement wrench is a consumable. Replace it once a year.
- Securely tighten the tip by using the tip replacement wrench. If the tightening force is weak, vibration may weaken or fail.

3-2 Output setting

- Use only at the Max power level described on the tip case or less.
- While observing the patient's condition, make sure to use from the lowest power setting, then increase power, according to each case.

Max. Power Level	
Varios2 Varios 970 570/370	Varios Varios 750 550/560/350
Ⓒ 10	Ⓒ Max.

* Check the model of the handpiece.

**CAUTION**

- Make sure to use the tip only at the max power level or less. If used at a level exceeding the max power setting level, the tip may break or the surface of tooth may be damaged.

* Make sure to read the "Varios Series" instruction manual and attached documents carefully before use.

3-3 How to use

- 1) Attach the tip securely (See 3-1 Installation.).
- 2) Set the output setting of the "Multi Function Ultrasonic Scaler Varios Series" to the max power level described on the tip case or less (See 3-2 Output setting.).
- 3) Adjust the water volume and the orientation of the tip apex to the surface of the tooth so that water injection passes through to the tip apex.
- 4) As a previous treatment for tooth removal, remove the periodontal membrane to create clearance for mobility.

* For operation of the "Multi Function Ultrasonic Scaler Varios Series", see the "Varios Series" instruction manual.

4 Application

Use for removing (rupturing) the periodontal membrane at the time of tooth removal.

Case 1: Stump tooth removal (upper left No. 2 tooth removal)

Points to remember: Since bone resorption easily occurs on the labial side, caution is required in the way of moving the tip apex so as not to damage the alveolar bone (Fig. 4). Attention should also be given to contact with the existing tooth.

Case 2: Angular impaction (lower left No. 8 tooth removal)

Points to remember: For treatment of an impacted wisdom tooth, be careful not to apply excessive force like an elevator. Also, for removing on the lingual side, move the tip while carefully observing inward inclination of the tooth, contour of the tooth crown, etc.

• How to use the G95 (tip for periodontal membrane removal) is introduced by video.
<http://www.nsk-dental.com/en/products/oral-hygiene/multifunction-ultrasonic-scalers.html>

CAUTION

- Being aware of the position of the tip apex portion, constantly move the tip to prevent it from staying at one point as best as possible, and intermittently apply pressure with care. If the tip is held in one place, which results in insufficient cooling, the bone may be overheated or the bone may be ablated excessively.

CAUTION

- Move not only in the mesiodistal and buccolingual directions but also up and down as if delivering water to the apex (light pumping action). At that time, pay attention to the reaches of water injection (Fig. 3).
- In order to prevent damage to the alveolar bone, move the tip apex along the tooth to the extent possible (Fig. 4).

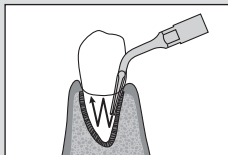
Fig. 3

Move the tip with an image of delivering water to the apex (light pumping action).

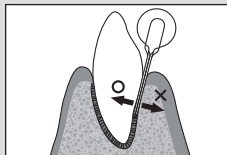
Fig. 4

In order to prevent damage to the alveolar bone, move the tip apex along a tooth to the extent possible.

- If the tip is operated too deeply or is held in one location for a prolonged period, postoperative pain, swelling, bone damage, loss, sequestration, etc., may occur due to thermal stimulation of the tip.
- Do not bring the tip into contact with soft tissue, such as gums or mucous membrane, for a prolonged period. The tissue may become burned.
- Do not apply force to the tip apex more than necessary. The tip may fracture, curve, etc. Pay extra attention in applying pressure in the lateral direction.
- Tooth removal cannot be completed with this tip alone. The tip for periodontal membrane removal serves as a complement to removal of the periodontal membrane; where socket damage is minimized by the rupturing periodontal membrane as a previous treatment for tooth removal. In accordance with general rules, make sure to use also an elevator, periotome, forceps, etc.
- Do not use the tip in place of an elevator, etc., while prying or turning the tooth. The tip may be deformed or fractured.

**CAUTION**

- Depending on the number of operating times, bone property, and tooth condition, the tip surface coating (TiN coating) may be removed. If the coating is removed, the tip surface will not glide well and the ablating ability will deteriorate. If such a symptom appears, replace the original tip with a new tip.

5 Maintenance

Time for tip replacement

Replace the tip if worn or deformed.

Sterilization

- After use, make sure to wash, sterilize, and then store the tip.
- Perform autoclave sterilization at temperatures up to 135°C. Example) for 20 minutes at 121°C or 15 minutes at 132°C.
- If the temperature rises above 135°C in a drying process, make sure to delete the drying process. Overheating could lead to early deterioration of the tip.
- Do not perform washing, immersion, or wiping with electrolyzed oxidizing water (strong acidic water or super acidic water) or sterile liquid.

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NSK Europe GmbH  www.nsk-europe.de
NSK United Kingdom Ltd www.nsk-uk.com
NSK France SAS www.nsk.fr
NSK Oceania Pty Ltd www.nskoceania.com.au
NSK America Corp www.nskdental.com
NSK Asia Pte Ltd www.nsk-inc.com